

Kangaroo™ Joey Enteral Feeding Pump

User Guide (Feed & Hydration Mode)

Pump Set Loading Procedure.




1. Locate power button on lower right hand corner of control panel. Press to turn pump on.



2. Confirm if you want to **KEEP** or **CLEAR** any prior pump settings.



3. With pump display facing you open blue door at top of pump.

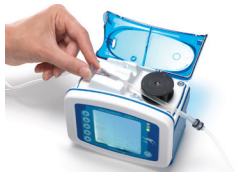

4. Grasp finger tab on valve and insert valve into valve slot on the left.




5. Grasp black ring retainer, and gently wrap tubing anti-clockwise around rotor. Pull up retainer and insert into slot.



6. Close blue door. Display should read **SET LOADED**.


Set Priming and Pump Programming Procedure.

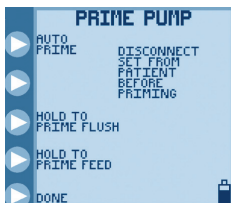


1. Press **PRIME PUMP** followed by **AUTO PRIME** to automatically prime or **HOLD TO PRIME** to manually prime both feed and flush lines. Press **DONE** when complete.




2. Press **ADJUST FEED** followed by **FEED RATE** to adjust hourly delivery rate.

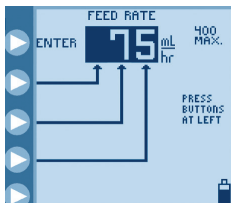


3. Press buttons on left to program hourly feed rate. Press **ENTER** to confirm rate.

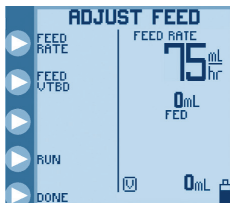

4. Press **FEED VTBD** to program feed Volume To Be Delivered if desired.

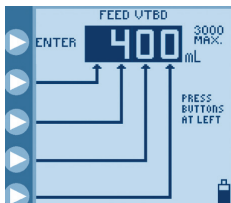


5. Press buttons on left to program volume of feed to be delivered. Press **ENTER** to confirm rate, followed by **DONE** to return to main screen.

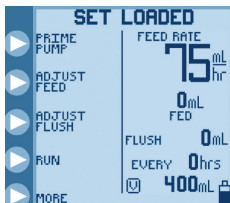


6. Press **ADJUST FLUSH** to program flush volume and interval.

POWER DOWN PROCEDURE


Locate power button on lower right hand

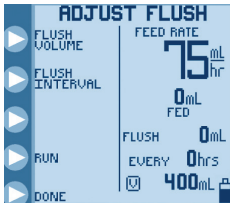

7. Press **FLUSH VOLUME** to program flush volume. Press **FLUSH INTERVAL** to program interval between each flush.

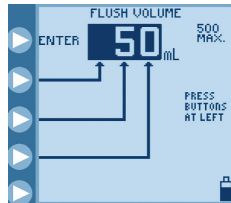

8. Press buttons on left to program volume of flush to be delivered. Press **ENTER** to confirm volume.

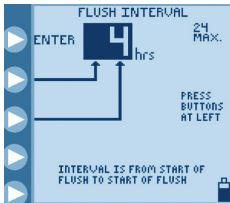

9. Press buttons on left to program flush interval. Press **ENTER** to confirm interval, followed by **DONE** to return to main screen.

10. Press **RUN** to commence feeding regime. In order to achieve proper accuracy, the initial fluid line in the feeding set container must be 15 cm above the top of the feeding pump when initiating the feeding cycle.



* **IMPORTANT NOTE.** Kangaroo™ Joey Pump will automatically flush an additional 25mL of water after completing the programmed delivery of formula, as long as at least one flush has occurred during formula delivery.

Alarm and Error Code Information.



HOLD ERROR

Pump left in Hold for more than 10 minutes.

Press CONTINUE followed by RUN to start pump running or power down pump.



ROTOR ERROR

Problem with pump set tubing around the rotor.

Ensure silicone tubing is correctly loaded around rotor.



FLOW ERROR

Obstruction in line downstream from pump.

Check for kinked or blocked tubing below pump and rectify.



FEED ERROR

Feed container empty or obstruction in the line upstream from pump.

Check for kinked tubing or obstruction in line above pump or attach new/refilled feed container.



FLUSH ERROR

Water container empty or obstruction in the line upstream from pump.

Check for kinked tubing or obstruction in flush line above pump or refill flush container.



FEEDING COMPLETE

Programmed feed has been delivered.


Power down pump.



PUMP SET DISLODGED

Black retainer ring (MISTIC) is not properly loaded in MISTIC pocket.


Ensure pump set is correctly loaded.

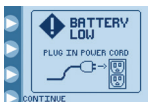

BATTERY LOW

Battery must be recharged immediately.

Plug power cord into AC outlet.

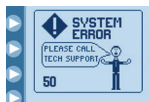

SYSTEM ERROR

General error - see pump manual.

Send pump for service.

For complete operation information please refer to Kangaroo™ Joey enteral feeding pump operation and service manual

cardinalhealth.com.au

Cardinal Health Australia

Level 2, 5 Eden Park Drive, North Ryde NSW 2113

